




How to disinfect face shields effectively

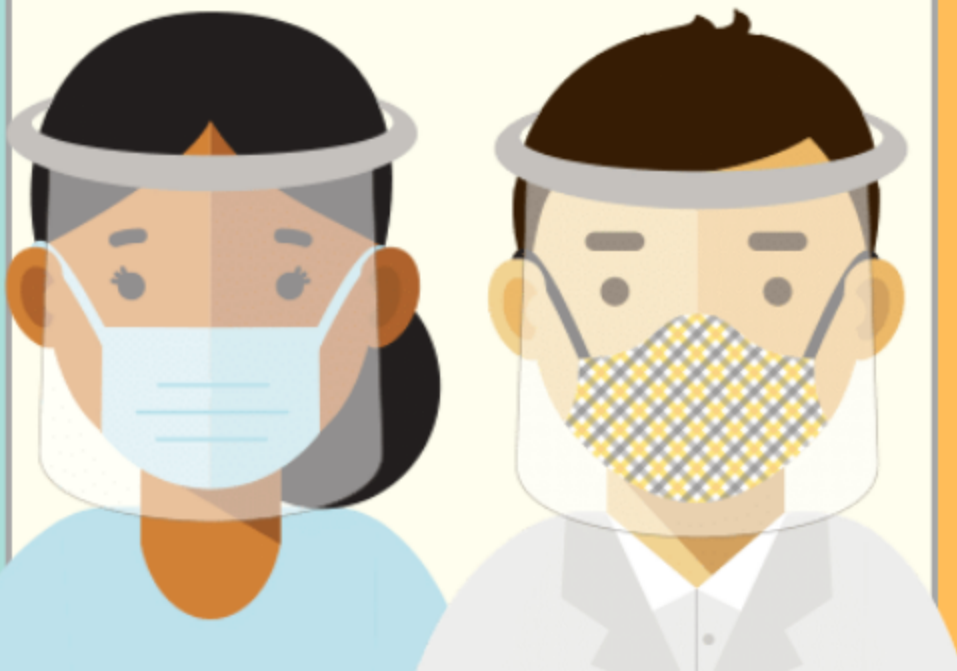

1. Cleaning the shield from the inside to outside by using wipes with cleanser.

2. Sterilizing the shield by using wipes with bleach, or any solution that contains chlorine 0.1%, twice in a round and pause between rounds.

3. Wiping first with pads or blowing on the shield until it is dry. Then, clean again with damp wipes.

4. Do not use the face shield if anybody coughs or sneezes on it. Please also discard the shield in a closed waste bin.

5. Using alcohol to clean the shield will change the look of the plastic.

